3-in-1 THERA°PEARL BREAST THERAPY

Hot or Cold Breast Therapy
Terapia Mammaia con frío o calor
Terapia a caldo o freddo per il seno
Warmte- of koudetherapie voor de borst
Thérapie mammaire par le chaud ou par le froid

		<table><tbody><tr><td>700 WATTS</td><td>16 SECONDS</td></tr><tr><td>1,000 WATTS</td><td>13 SECONDS</td></tr><tr><td>1,250 WATTS</td><td>11 SECONDS</td></tr></tbody></table>	700 WATTS	16 SECONDS	1,000 WATTS	13 SECONDS	1,250 WATTS	11 SECONDS
700 WATTS	16 SECONDS							
1,000 WATTS	13 SECONDS							
1,250 WATTS	11 SECONDS							

BF *Breastfeeding is the most natural gift you can give your baby. However, as natural as it is, breastfeeding sometimes presents challenges. Lansinoh® brings you advanced relief with TheraPearl 3-in-1 Breast Therapy, reusable and non-toxic packs that are latex, lead and BPA free and can be used hot or cold for a range of conditions.*

Intended Use: Use Lansinoh® Therapearl hot to help relieve plugged ducts and mastitis, and when engorged, just before feeding to help let-down and milk flow. It can also be used before or during pumping sessions to reduce the total time spent expressing by encouraging a faster let-down. Use Lansinoh® Therapearl cold to help provide relief from pain and swelling due to engorgement or mastitis. To hold them in place, packs can be placed in a bra.

For heat therapy: Place one pack inside purple cloth cover and microwave lying flat for time indicated in the table (700 Watts – 16 seconds/ 1,000 Watts – 13 seconds/ 1,250 Watts – 11 seconds). Each pack should be heated separately. After testing pack temperature, you may add more time in 5 second increments as needed until desired temperature is reached.

For cool therapy: Freeze for at least 2 hours then place into covers before use.

For use with a pump: Heat packs according to heat therapy instructions and wrap around the outside of breast pump funnels, securing with the fastener, while pumping.

- For external use only - do not use for purposes other than those stated above
- Lansinoh® Therapearl should only be applied for maximum 20 minutes
- Clean with a mild soap, detergent or alcohol
- Do not place any heavy objects on top of the packs
- Do not use on infants or allow infants to chew on the packs

CAUTION: Do not heat Lansinoh® Therapearl packs beyond the time stated. Heating beyond suggested time may damage or destroy Therapearl pack. Always test pack temperature prior to application. Overheating or prolonged application may cause injury. Do not apply to sensitive skin or open wounds. Use only under adult supervision. Do not puncture pack and/or ingest contents. Do not fall asleep using pack. For extreme, prolonged or chronic pain, consult a physician.

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Scan with smart phone for product information.

TH10400IF1013

Lansinoh Laboratories, Inc. Jason House, Kerry Hill, Leeds, LS18 4JR, UK

Lansinoh®, HPA® and Affinity® are registered trademarks of Lansinoh Laboratories, Inc. TheraPearl® is registered trademark of Therapearl LLC ©2013 Lansinoh Laboratories, Inc. All rights reserved. Patent(s) and/or design registrations issued or pending: See www.therapearl.com/patent for specific details.

		EMERGO EUROPE Molestraat 15 2513 BH, The Hague The Netherlands Fax: +31 (0)70 346 7299	TheraPearl 8106 Stayton Drive Jessup, MD 20794-9615, USA www.therapearl.com	Made in China

PUMP

2 in 1 Electric Breast Pump

- Use as a single or double pump
- 3 pumping styles you can tailor
- Easy to use and clean

Manual Breast Pump</