GB In the early days after birth, mothers (may) often experience engorgement or swollen breasts. This may result in the nipples temporarily becoming flat or inverted, making it difficult for the baby to breastfeed easily. Lansinoh Latch Assist™ is a simple tool that enables you to temporarily draw out the nipple so that your baby can properly grasp the nipple and areola in order to enable a good latch.

Directions for use

Before use, wash Latch Assist™ in hot, soapy water. Rinse and air dry.

Select the Latch Assist™ size that is most comfortable for you. In order to decide which one is best, check how your nipple fits into the shield. If your nipple rubs along the sides of the shield, it is too small. If there is too much space between your nipple and the sides, it is too large.

With your baby in position to breastfeed, gently squeeze bulb and place the shield over and around your nipple. Lightly press the shield down against the skin to ensure an airtight seal. Slowly release the bulb for the desired pressure. Repeat gently until your nipple is extended and your baby can easily latch on to the breast. After use, wash again, rinse and air dry. Discard any milk that may collect in bulb.

Please note that for hygiene reasons the Latch Assist™ is a single user product.

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Lansinoh® is a registered trademark of Lansinoh Laboratories Inc.
©2012 Lansinoh Laboratories Inc. All Rights Reserved