Lansinoh®

**PUMP**

- 2 in 1 Electric Breast Pump**
- Use as a single or double pump
  - 3 pumping styles you can tailor
  - Easy to use and clean

- Single Electric Breast Pump**
- Quiet and efficient
  - Two separate phases
  - Adjustable suction settings for comfort

- Manual Breast Pump**
- Separate let-down and expression phase
  - Gentle and efficient pumping
  - Easy to assemble, use and clean

**STORE**

- Breastmilk Storage Bags**
- Pre-sterilised and leak-proof
  - Double zipper for a secure seal
  - Write on tab above the fill area

- Breastmilk Storage Bottles**
- Wide neck bottles compatible with all Lansinoh® Breast Pumps and NaturalWave™ teats
  - BPA free

**FEED BREASTMILK**

To help maintain established breastfeeding patterns:

- Feeding Bottles with NaturalWave™ teat**
- Enables seamless transition from breast to bottle and back again
  - Promotes the natural sucking style of babies
  - Based on over 50 years of research

**BREAST CARE**

- Lansinoh® HPA® Lanolin**
- Helps soothe and protect sore and cracked nipples
  - 100% natural
  - No need to remove before breastfeeding
  - No preservatives, antioxidants or other additives

- TheraPearl® 3-in-1 Breast Therapy**
- Use hot to relieve mastitis, plugged ducts and enhance milk flow
  - Use cold to soothe breast soreness and relieve swelling due to engorgement
  - Use with a breast pump to encourage milk let-down

- Lansinoh® Latch Assist™**
- Provides gentle suction in order to draw out the nipple to assist with latch on
  - Easy to use
  - 2 cone sizes for a comfortable fit
  - BPA free

- Ultra thin, stay dry Nursing pads**
- Discreet, naturally contoured
  - Soft and breathable for comfort
  - Non-slip tapes for secure fit
  - One-way moisture technology

**A natural and organic baby toiletries range**

- ✓ With organic & natural ingredients
- ✓ Gentle blend for baby's sensitive skin
- ✓ With moisturising Aloe Vera
- ✓ No artificial fragrances or colours
- ✓ Free from any nasties like SLS or Parabens
- ✓ No animal testing
- ✓ Dermatologically approved

Lansinoh  
**mOmma®**

**Playful & engaging**  
Cups and spoons rock to entertain baby, making mealtime more enjoyable

**Encourages baby's development**  
Colours, shapes and textures engage baby and stimulate learning

**Safe and hygienic**  
Protective covers ensure optimum hygiene and are perfect for "on the go" occasions

Lansinoh Laboratories, Inc.

Jason House, Kerry Hill, Leeds, LS18 4JR, UK

Lansinoh®, HPA®, mOmma® and earth friendly baby® are registered trademarks of Lansinoh Laboratories, Inc.

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www.lansinoh.co.uk

Scan with smart phone for product information.

BG40055/44204IF0114

BREASTFEEDING EXPERTS FOR OVER  
**30 YEARS**

## GB Breastmilk – best for your baby

The milk you express for your baby is a precious fluid. It combines the best possible nutrition with antibodies, live cells, and other agents that protect babies from infection and help them grow and develop. You'll want to take good care of it and how you store it affects how well it is preserved.

Lansinoh® breastmilk storage bags are made from food safe polyethylene (PE). Lansinoh's special manufacturing process means that they contain no additives and no plasticisers, so there is no risk of these substances getting into your breastmilk.

Please handle Lansinoh breastmilk storage bags with care when storing in and removing from the freezer, and especially when defrosting. If a bag is dropped, or damaged in any way, it may leak while defrosting.

### Instructions for use

- Tear off the safety seal at the top of the bag and open by gently pulling the zipper apart.
- Lansinoh breastmilk storage bags are presterilised and do not need to be washed before use.
- Pour breastmilk into the storage bag.
- Before closing the bag, read the fill quantity by means of the imprinted scale, close the bag and mark quantity in the inscription area.
- Bags can be stored upright or flat in the fridge or freezer. When stored flat, bags are easier to thaw.
- To avoid freezer burn, as much air as possible should be removed from the storage bags before freezing. To do this, take the bag between your forefinger and middle finger (like scissors) and press the air above the liquid level towards the zipper.
- Seal the bag by compressing the double seal. Repeat the procedure to ensure safe sealing.
- Single use only.

### Storage of breastmilk

Check with your midwife, health visitor or breastfeeding counsellor for specific storage instructions. When you are freezing your breastmilk, do not fill containers more than ¾ full to allow for expansion from freezing. Date containers with the date you expressed your milk for future reference.

How long can breastmilk be stored?	
Room temperature	6 to 8 hours
Fridge (up to 4° C)	Up to 3 days
Freezer 0° F (-18° C)	6 months

How long can thawed breastmilk be stored?	
Fridge (up to 4° C)	unopened 24 hours
Fridge (up to 4° C)	after opening 12 hours

### Notes

DO NOT refreeze thawed breastmilk. DO NOT store breastmilk in freezer door. The door tends to be the warmest part of the freezer. Date all breastmilk and use the oldest breastmilk first.

### Instructions for defrosting

To preserve the precious substances contained in your breastmilk, defrosting should be carried out as gently as possible – either very slowly over 24 hours in the fridge, or at room temperature. Do not use boiling water or a microwave! If necessary, you can defrost your breastmilk under running cold or lukewarm tap water or in a bottle heater (max 37° C). After defrosting, you can put the milk back into the fridge until consumption. Unopened, the bag containing defrosted breastmilk can be kept in the fridge for 24 hours. After opening, defrosted milk must be kept in the fridge and consumed within 12 hours. After warming, the remainder must be discarded. If the milk settles into different layers, you can mix it by carefully shaking the bag.

### Other Lansinoh products:

- Lansinoh® HPA Lanolin – for sore nipples and cracked skin
- Lansinoh® Nursing Pads – ultra thin, stay dry
- Lansinoh® TheraPearl 3in1 Hot or Cold Breast Therapy
- Lansinoh® Latch Assist™
- Lansinoh® Manual Breast Pump
- Lansinoh® Single Electric Breast Pump
- Lansinoh® 2in1 Electric Breast Pump

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All rights reserved.